Français

English

Manuel / Manual

 EUROFLEX®
Flexmobil i6

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INTRODUCTION

We congratulate you to the choice of Flexmobil i6 as your power wheelchair!

We will ensure that you will be satisfied with this Euroflex product from product from Eurovema Mobility AB, constructed and build in Sweden. Flexmobil i6 is built to meet very high standards of ergonomics, seating comfort and function.

Read the instructions carefully to benefit for to benefit your Flexmobil in 6 all possibilities.

USERS AND USE

Flexmobil i6 is an electric indoor wheelchair.

It is intended for children and adults with limited mobility. Persons who are able to control and use the interface of a powered wheelchair. It is strongly suggested to have the use of the wheelchair evaluated by a professional physiotherapist or doctor or other qualified and trained staff. Flexmobil i6 is available with different seating sizes that have a variety of settings and are easy to adapt individually for every need. The expected design life: 10 years.

TEST SPECIFICATION

Flexmobil i6 is the CE mark and meet the requirements of Law 1993: 358 (93/42 / EEC) on medical devices and the MPA procedure for such medical devices, LVFS 2013: 11. It meets all the requirements of EN 12184:2014 Class A.

CONTACT

If you have questions or need help with your product, please contact the local agent.

To get in touch with the manufacturer see info below:

Eurovema Mobility AB

Tel: +46-371-390100

E-mail: info@eurovema.se

Baldersvägen 38

332 35 GISLAVED

SWEDEN

ASSOCIATED DOCUMENTS

Associated product data sheets, service manual, installation instructions, safety notices and possible product recalls are available to read or download at www.eurovema.se

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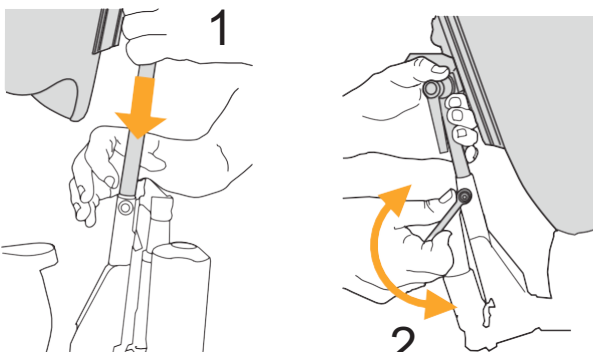
**ASSEMBLY**

- Open the packing and check that no transport damage has occurred.
- Check that the delivery harmonizes with the order.
- The chair is delivered with backrest and armrest not mounted!
- Press in the spring ball and slide in the back support pipe as shown in the picture (1). Tighten the handle (2) and the safety screw on the side of the back support pipe (3, 4).

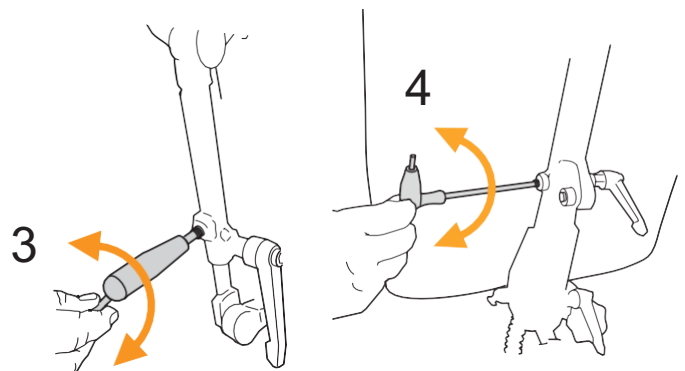
TRANSPORTATION

When transporting Flexmobil in a motor vehicle it is important that the brakes are engaged. See section "Disengagement of brakes". The chair should be secured with tie down straps. Attachment loops are available as standard.

Sitting in the wheelchair during the transport is strictly forbidden. Reduce the transport size by removing the armrest, backrest and the leg rest. The batteries in the wheelchair are sealed and maintenance free (AMG) and approved for transport in aircraft's. Further on transport see page 35.

Check at delivery that,

- all knobs and screws are tightened.
- the freewheel device for the brakes are deactivated.
- the arm- and backrests are raised and do not touch the hood when the seat is in the lowered position.
- the display shows no error codes (no flashing)
- battery is fully charged
- the joystick steering works in all directions.
- brakes works when you release the joystick.

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SAFETY

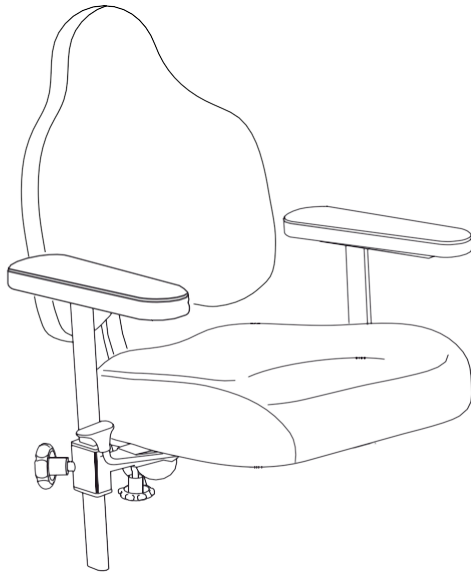
- Carefully read the manual before using the wheelchair.
- Charge the batteries when battery indicator shows orange.
- The lift motor has a duty cycle of 2 on /18 off meaning 2 minutes use followed by 18 minutes of rest.
- The wheelchair is designed to be used on in normal indoor climate.
- Be careful when adjusting the manual seat angle when seated. Could cause a risk of falling out of the chair.
- Immediately contact service if damages, i.e. loose parts or change in function of the chair is discovered.
- Use of the different seat settings can affect stability of the wheelchair. Use these only when the wheelchair is on a flat surface.
- Make sure to tighten all screws, knobs and levers after adjustments.
- Metal surfaces can become very hot if they have been exposed to sunshine or radiated heat.
- Service, maintenance and adaptations should be performed by educated staff, authorized of Eurovema Mobility AB.
- Use only original parts from Eurovema Mobility at repair.
- Do not exceed maximum user weight 150kg.
- Use only original charger for charging of the batteries.
- For maintaining safety use only original fasteners to the removable parts i.e. Armrest, backrest and foot-plate.
- The wheelchairs function may be disturbed in strong electromagnetic fields radiated by the e.g. power lines and data centers. The wheelchair itself can disturb the equipment based on electromagnetic fields such as alarm systems in shop, automatic doors, etc.

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
3

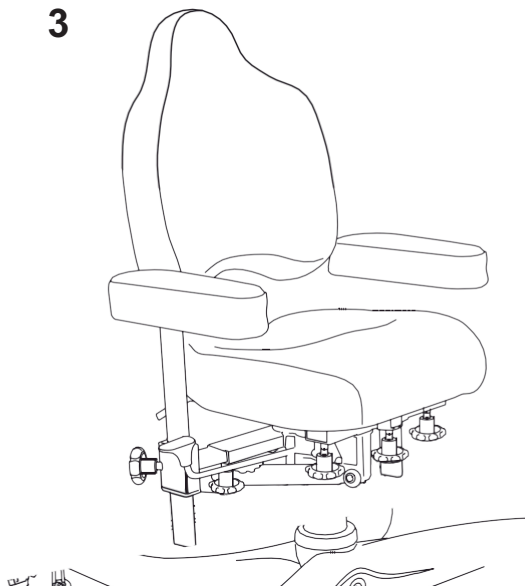
English

The Euroflex seating system is designed to optimize the seating comfort to the user. The upholstered seat cushion is available in different sizes to provide optimal seating comfort and support to the user. The cloth is made of dirty resistant and machine washable polyester.

Available seating combinations 1. SittRite 2. Comfort and 3. Childseat ABC.

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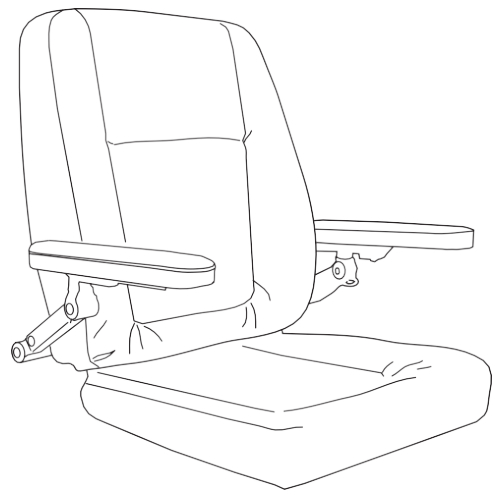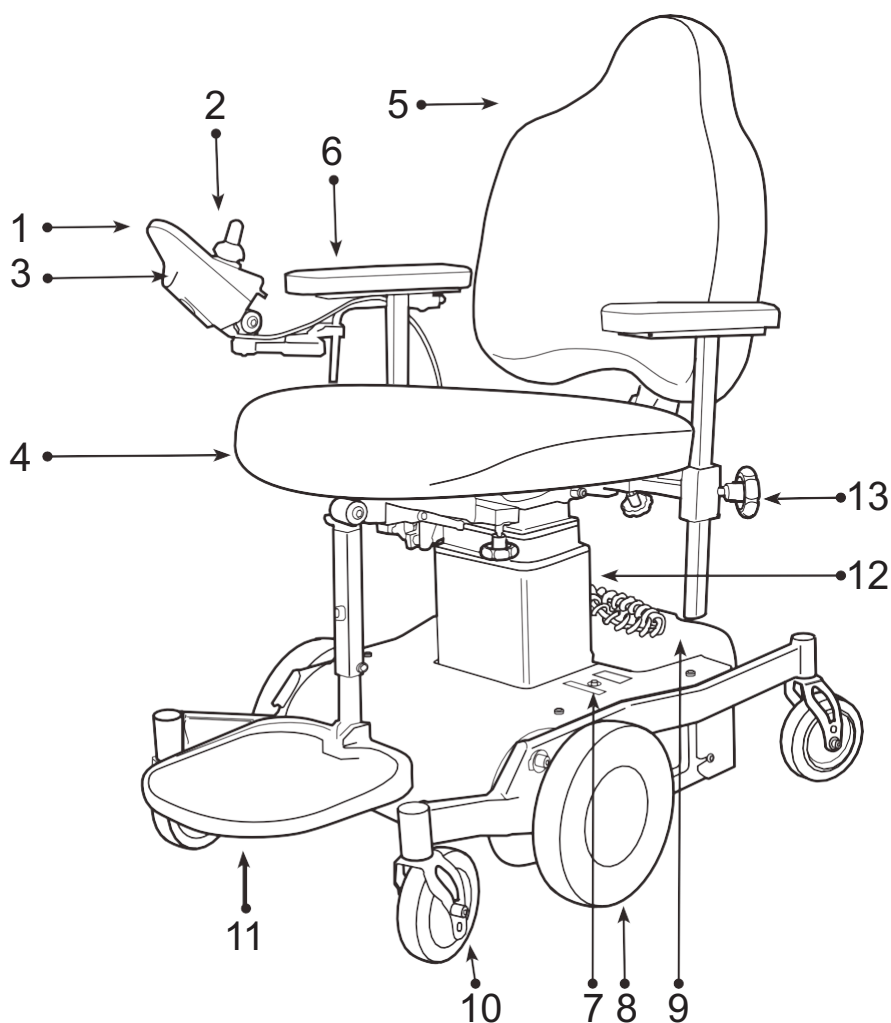
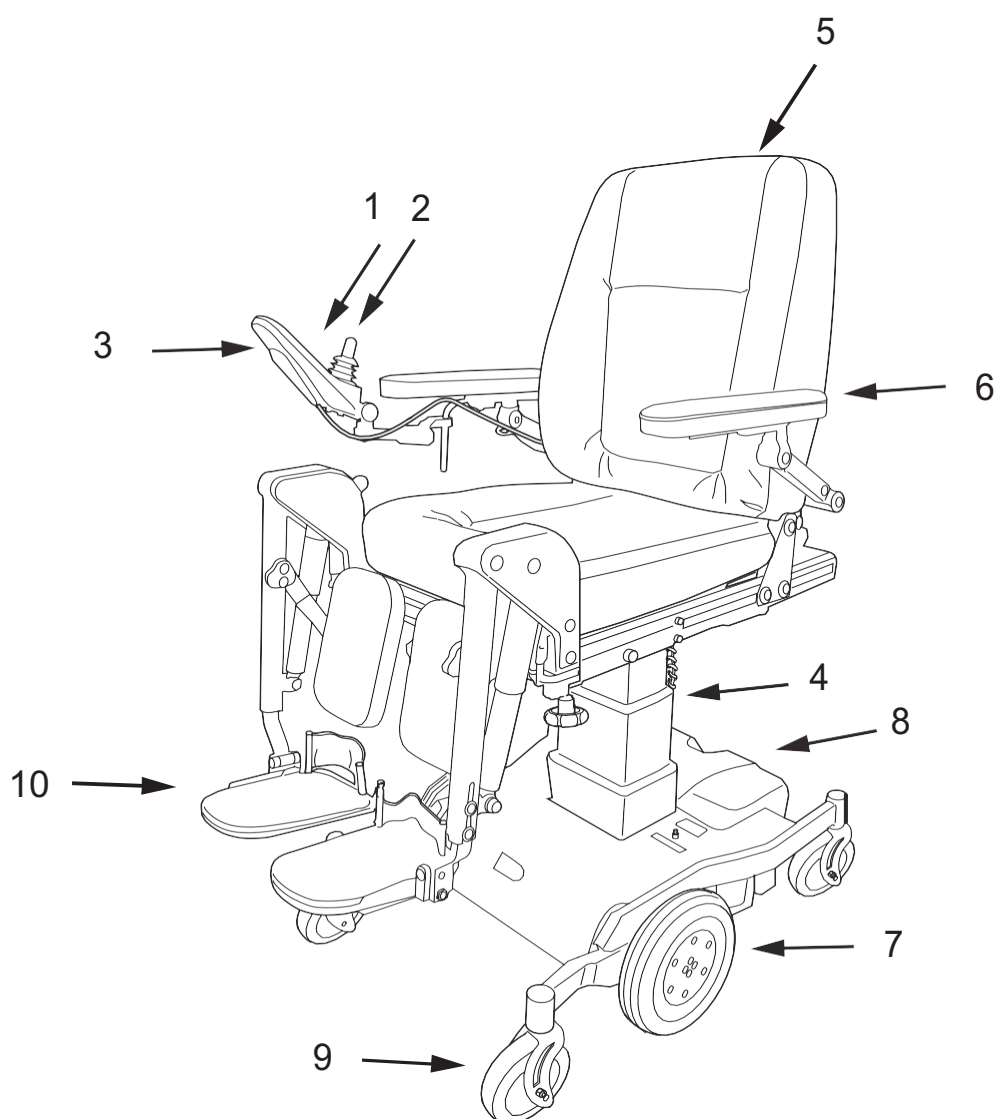
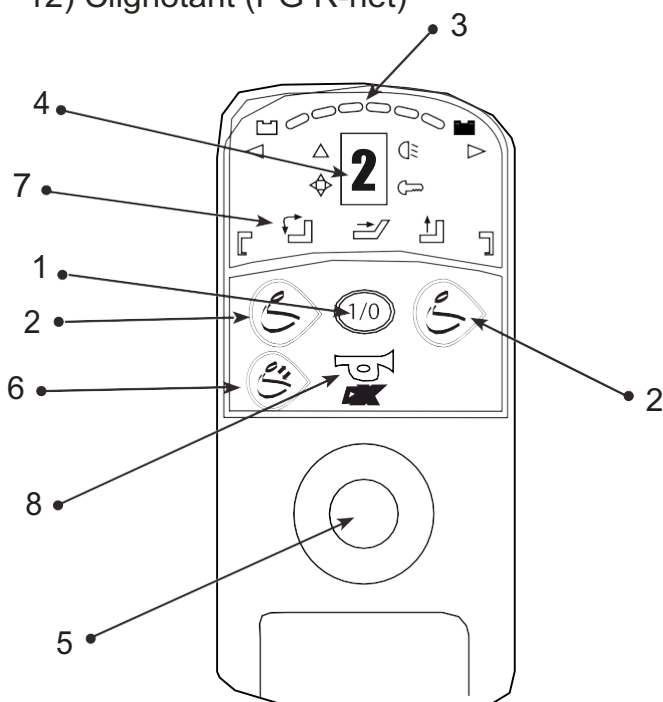
G90A

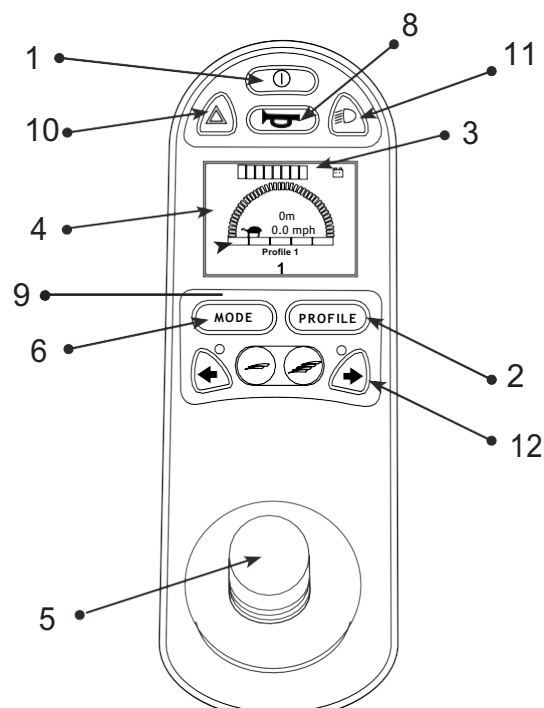
ELECTRONICS G90A and R-net

The Flexmobil i6 is equipped with Dynamic DX or PG Drives control system to control the power from the batteries out to the motors.

The wheelchair and its electric seat functions is maneuver by the control box, available in two different models; G90A and R-net. The joystick is for driving the wheel chair in the desired direction. The electronics can be programmed to suit the users needs but usually the original program is fully sufficient.

Should a fault occurs with the electronics the status LED (8) will blink. Thru counting the number of blinks you can deduce the problem. See chapter "fault trace".

- 1) On/off
- 2) Choice of drive programs
- 3) Battery indicator
- 4) Display for drive programs/speed
- 5) Joystick
- 6) Seat functions
- 7) Symbols for seat functions
- 8) Horn
- 9) Status led (pg r-net)
- 10) Hazard light (pg r-net)
- 11) Headlight (pg r-net)
- 12) Turn indicator (pg r-net)

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ELECTRONICS shark (option)

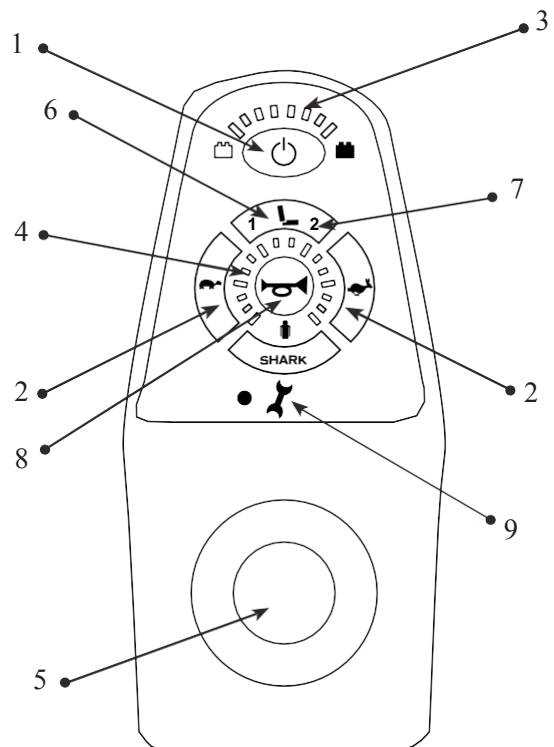
The Flexmobil i6 is equipped with dynamic control system to control the power from the batteries out to the motors.

The wheelchair and it's power assisted seat functions is maneuver by the control box, dynamic shark.

The joystick is for driving the wheel chair in the desired direction. The electronics can be programmed to suit the users needs but usually the original program is fully sufficient. Should a fault occurs with the electronics the status led (9) will blink. Thru counting the number of blinks you can deduce the problem. See chap. Fault trace.

- 1) On/off
- 2) Choice of drive programs
- 3) Battery indicator
- 4) Display for drive programs/speed
- 5) Joystick
- 6) Seat functions
- 7) Symbols for seat functions
- 8) Horn
- 9) Fault indicator

* The operating manual for the control unit is delivered separately with the wheelchair.

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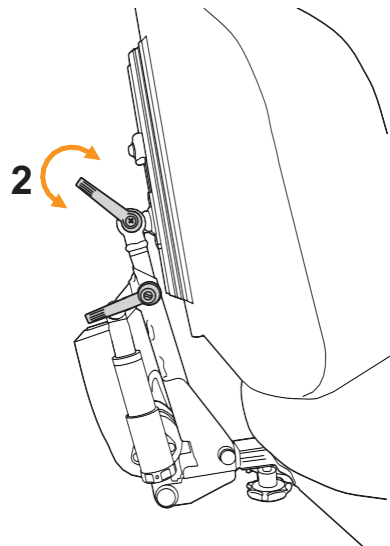
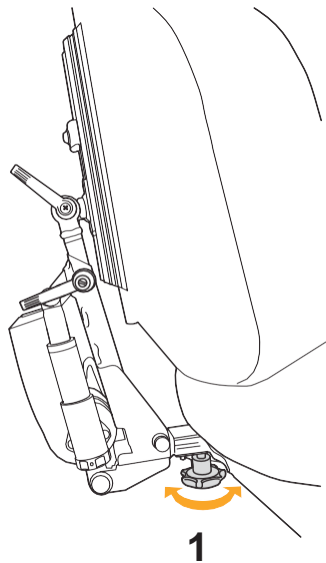
English

BACK REST - seat depth

To adjust the seat depth you can move the back support forward or backwards. Use the wheel nut (1). Adjust the desired seat depth by moving the back support forward or backwards. Lock by tighten the knob.

BACK REST - angle

To adjust the angle on the back support, loosen the lever (2). Adjust to desired angle and lock by moving the lever back to its original position.

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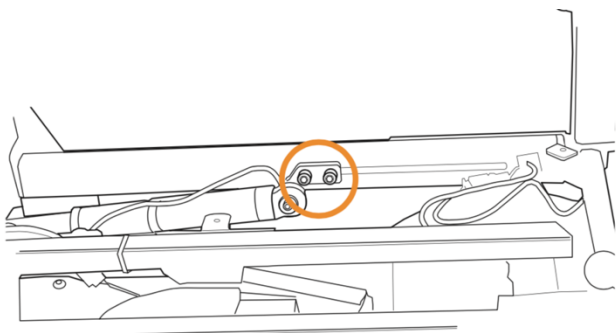
English

BACK REST Comfort - seat depth

To adjust the seat depth with back support loosening the two screws on the actuator and the four screws that hold the backrest of the seat frame. Adjust the desired seat depth by moving the backrest forward or back in the seat frame. Tighten all screws.

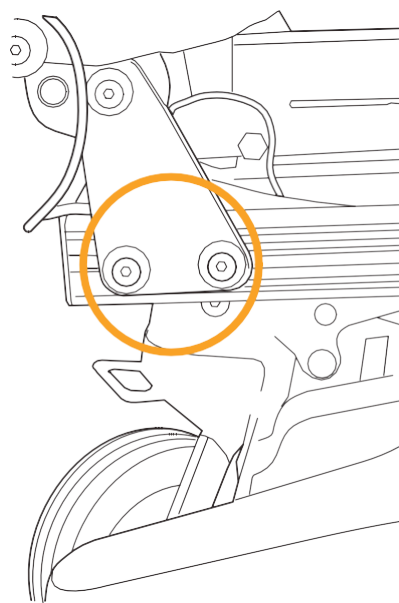
BACK REST Comfort - angle

To adjust the angle of the backrest and use the control box.

See picture on control box on page 9.

Dynamic Control box, press the red button to get to the seat functions. Choose a seat tilt by moving the joystick right or left, until the symbol for the seat tilt is on. Run the seat tilt forward, by moving the joystick lever forward and backward by moving the joystick lever back.

Dynamic Control box, press the MODE button to access the seat functions. Choose a seat tilt by moving the joystick right or left, until the seat tilt is highlighted in the display. Run the seat tilt forward, by moving the joystick lever forward and backward by moving the joystick lever back.

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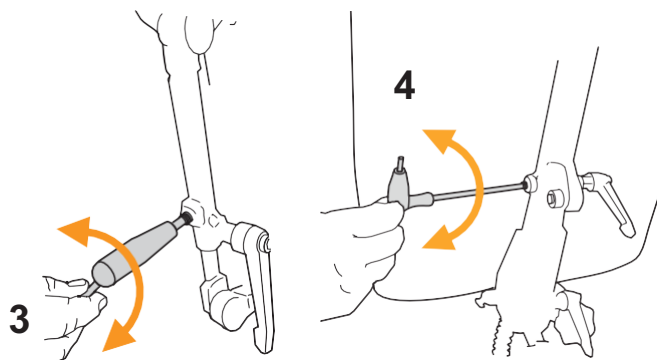
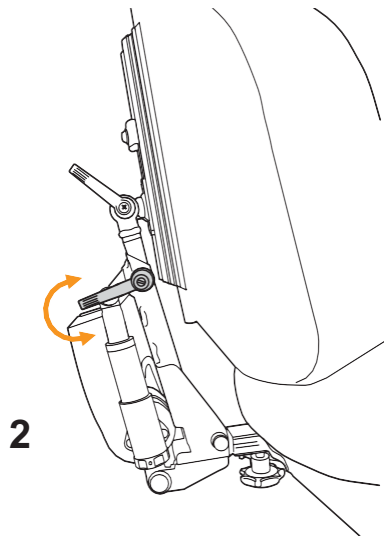
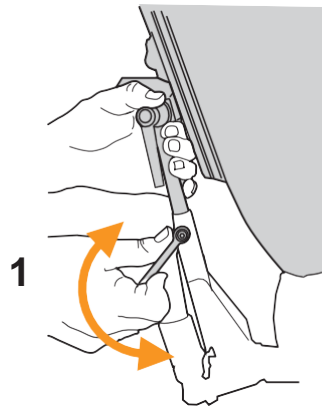
BACK REST - height

Loosen the security screws (3, 4) with an allen key before adjusting the high of the backrest (1). Loosen the lever (2) by turning it anti-clockwise 1/2 turn. Adjust to desired height of the back support and lock turning the lever clockwise 1/2 turn.

After adjustment tighten the security screws (3, 4).

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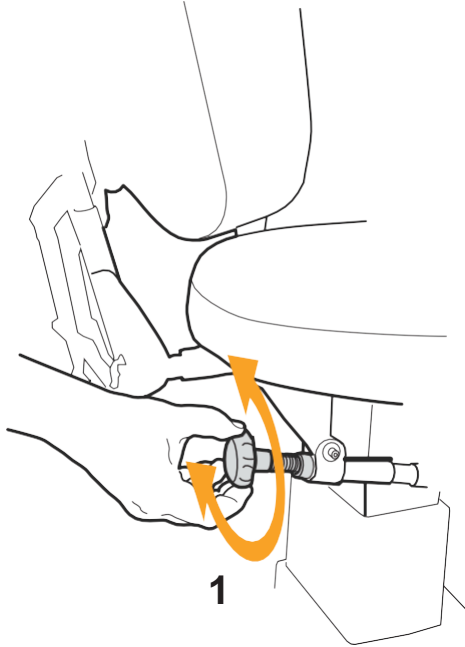
SEAT ANGLE - manual

The seat angle is possible to adjust from -14° to $+23^{\circ}$ backward. Turn the knob clockwise to tilt the seat forward and counterclockwise for back-ward (1).

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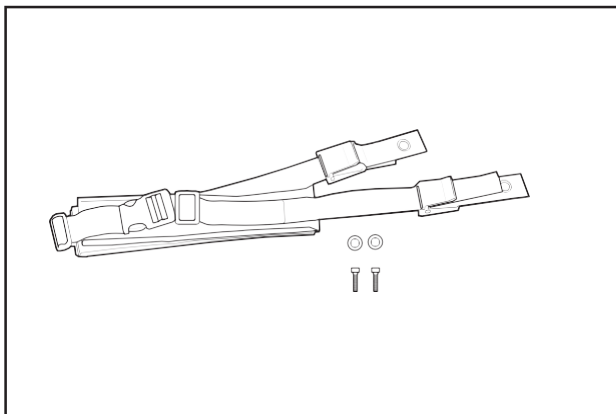
Additional pelvic support

NOTE! For correct support for the pelvis and stability to the torso, we recommend using seating system SitRite together with with trunk and thigh support.

The seat has an ergonomic built-in position cushion that prevents the sliding and provides optimal pelvic positioning. Which means that the pressure moves from the pelvis to the thighs.






This results in lower pressure on vulnerable areas.

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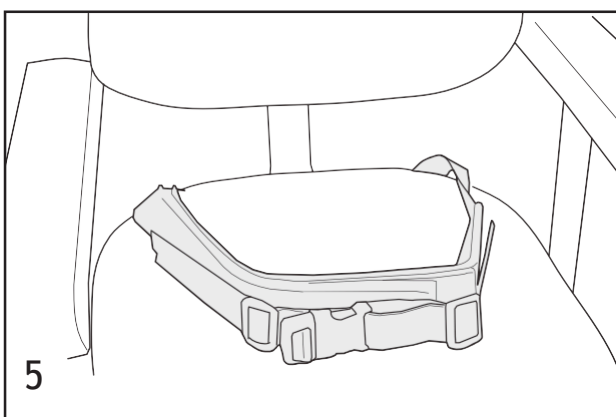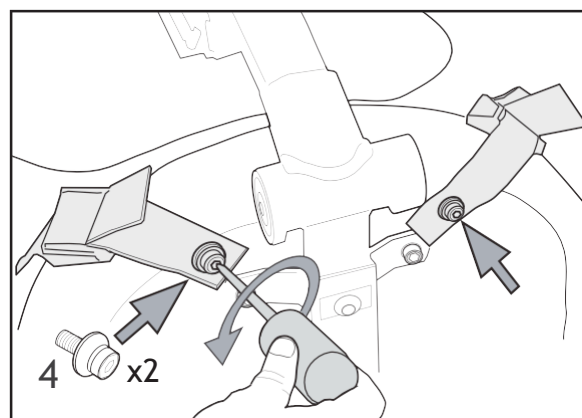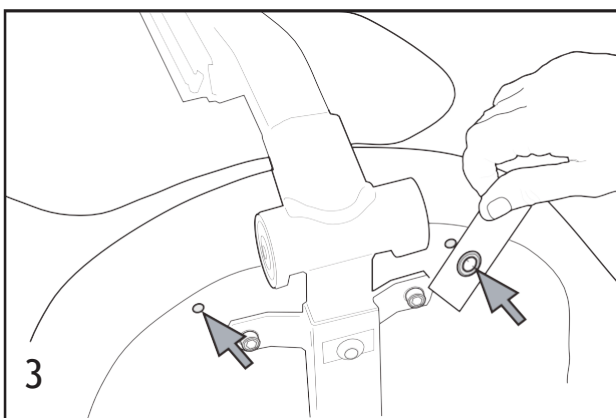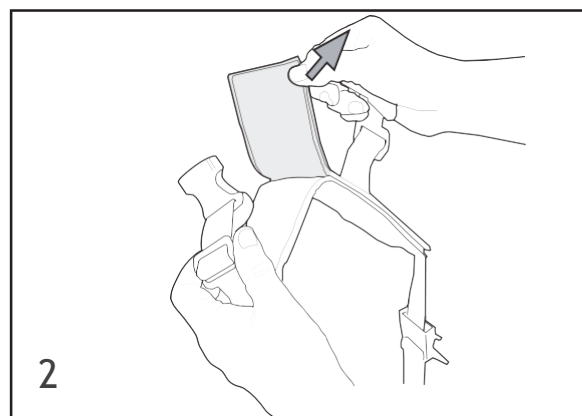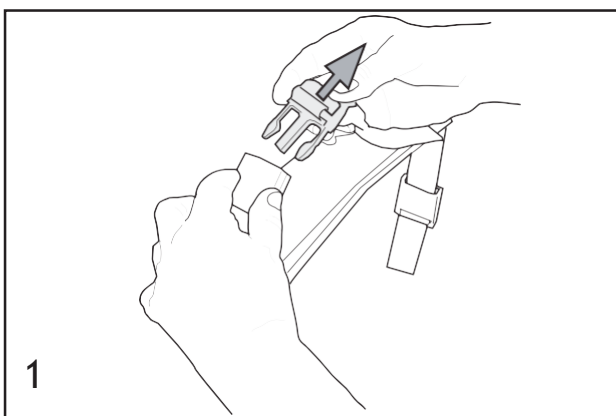English

SEAT HEIGHT

The seat can be raised stepless at any height and will be automatically locked in position.

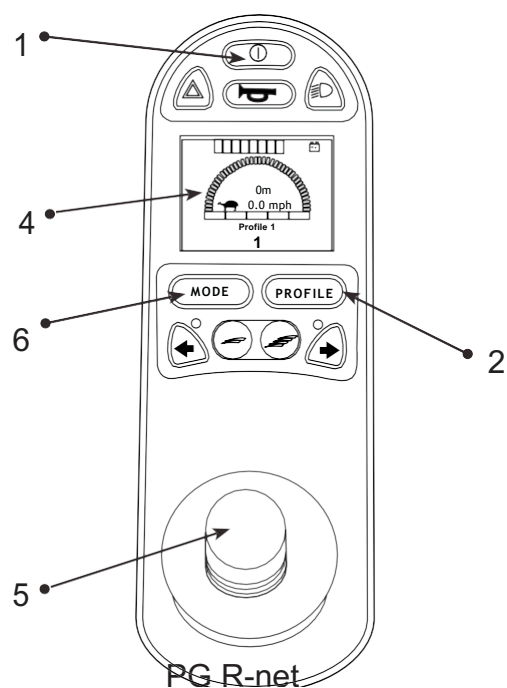
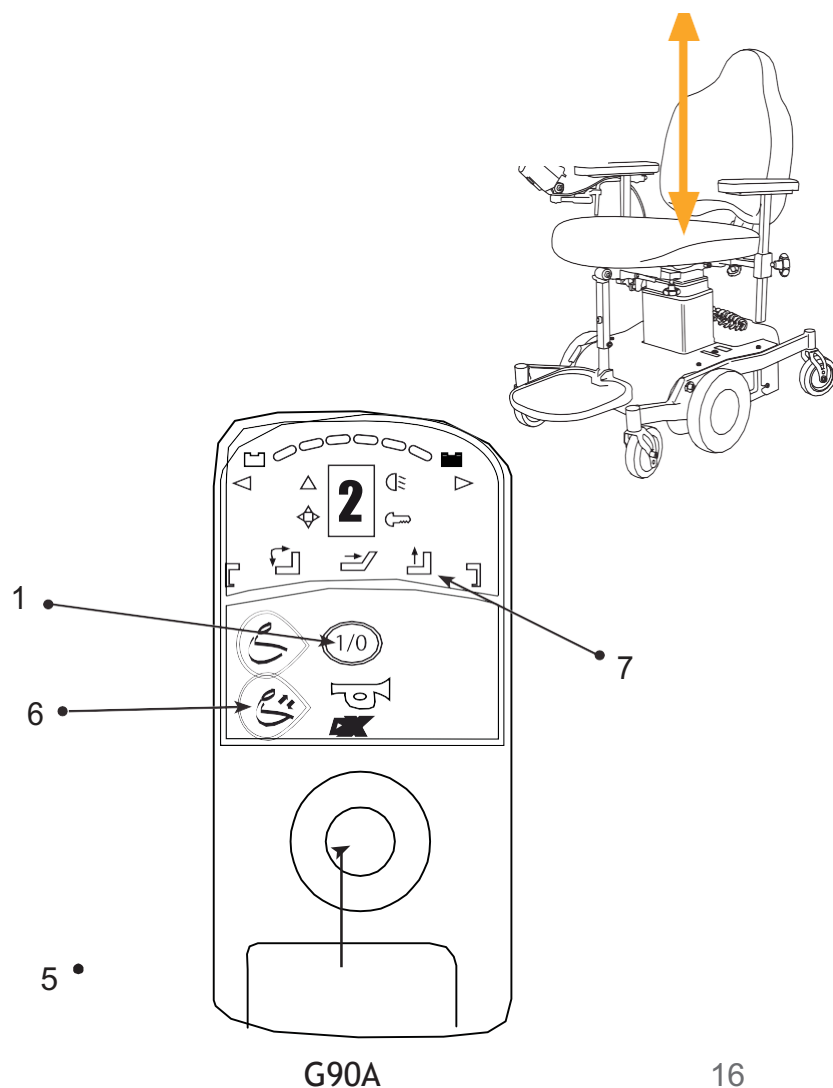
Start the electronics of the wheel chair by pushing button (1) on the control box.

G90A:

Choose the seat lift symbol (7) by pushing the red button several times (6) or moving the joystick (5) to the side until the "seat lift" symbol on the right blink.

PG R-net:

Press the "MODE" button (6) and move the joystick to the side and choose the "seat lift" symbol, shown in the display (4). Move then the joystick (5) forward to rise or to backwards to lower the seat height. The height adjustment stops automatically. Push button "PROFILE" (2) to get back into drive mode and the chosen drive program will be visible in the display (4).

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SEAT ANGLE - power

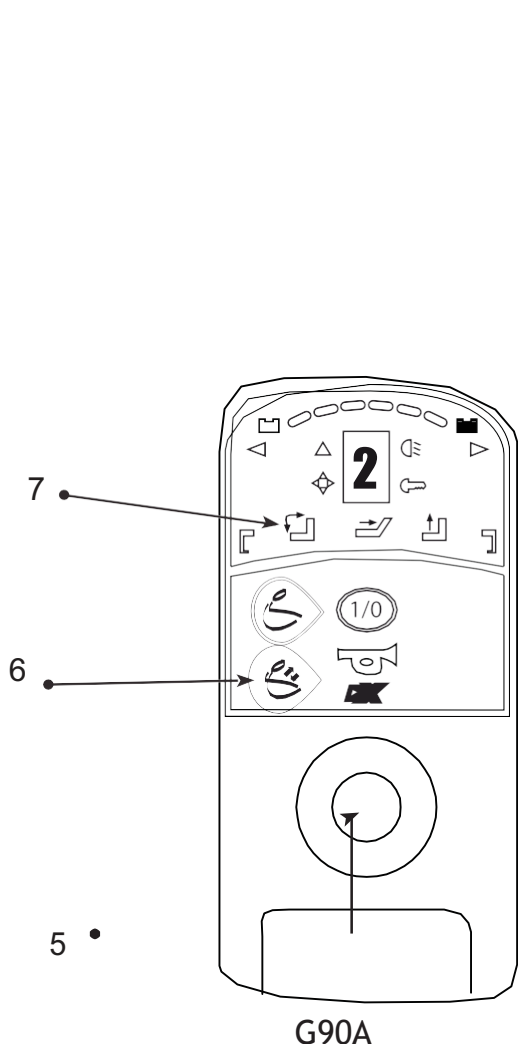
If the chair is equipped with power assisted seat tilt, it's activated by the control box.

G90A:


Choose a seat symbol (7) for the "seat angle" by pushing the red button (6) and move then the joystick (5) to the side until the "seat angle" symbol in the middle blink.

PG R-net:

Press the "MODE" button (6) and move then the joystick to the side and choose the "seat tilt" symbol, will be visible on the display (4). Move the joystick forward to move the seat tilt forward and backwards for tilt backwards. The seat tilt adjustment stops automatically. Push button "PROFILE" (2) to get back into drive mode and the chosen drive program will be visible on the display (4).

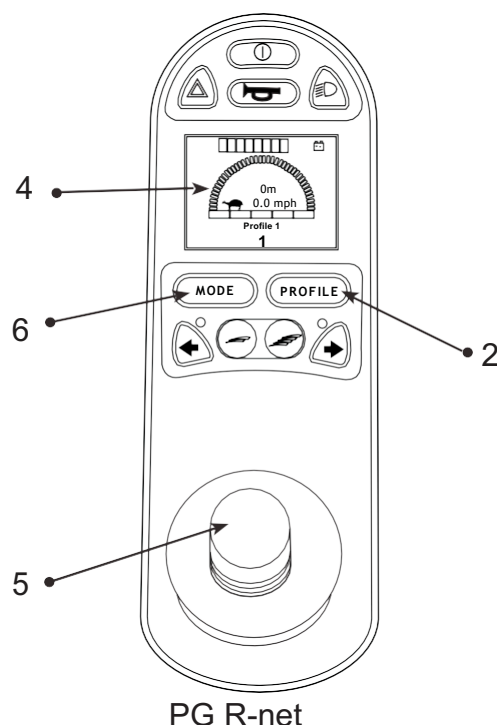ATTENTION POINT DE PINCEMENT!
CAUTION PINCH POINT!

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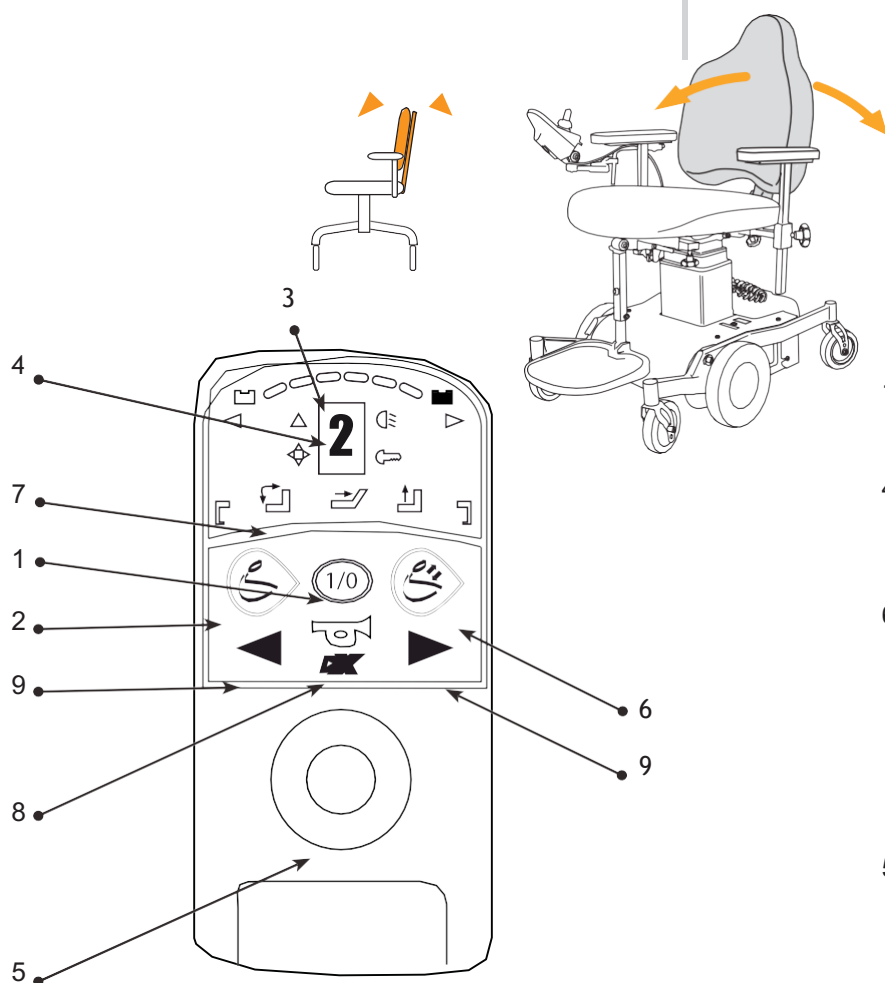
English

BACKREST ANGLE - power

If the chair is equipped with power assisted backrest its activated by the control box.

G90A:

Choose a seat symbol (7) for the “seat angle” by pushing the red button (6) and move then the joystick (5) to the side until the “seat angle” symbol in the middle blink.

PG R-net:

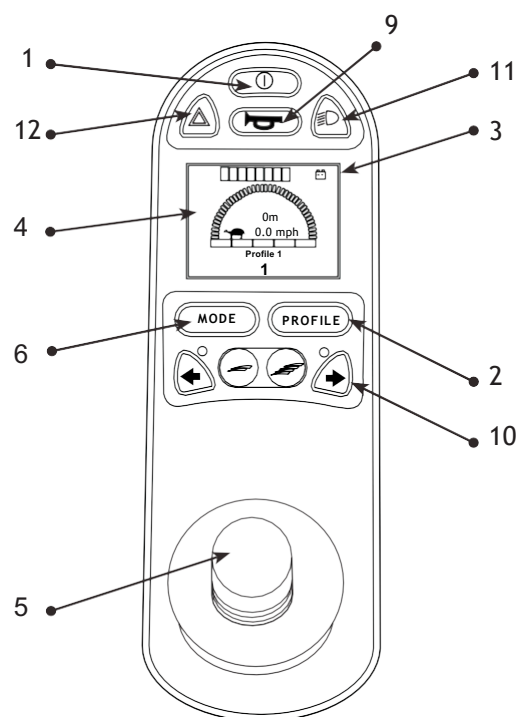
Press the “MODE” button (6). Move joystick to the right and choose the “seat angle” symbol (7). Move the joystick forward to move the backrest forward and backwards for backwards.

The seat tilt stops automatically when the endpoints is reached. Push button

“PROFILE” (2) to get back into drive mode and the chosen drive program will be visible in the display (4).

ATTENTION POINT DE PINCEMENT!
CAUTION PINCH POINT!

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

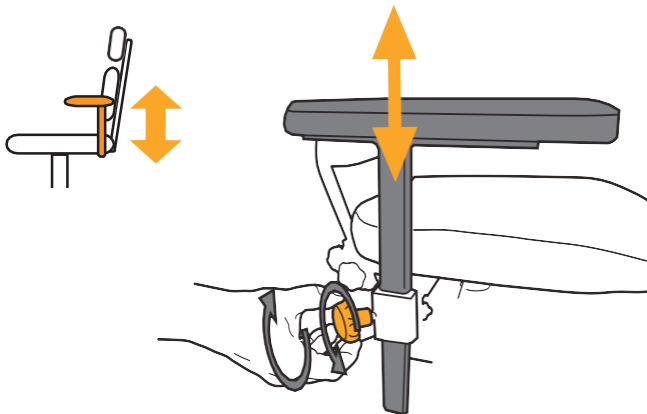
2

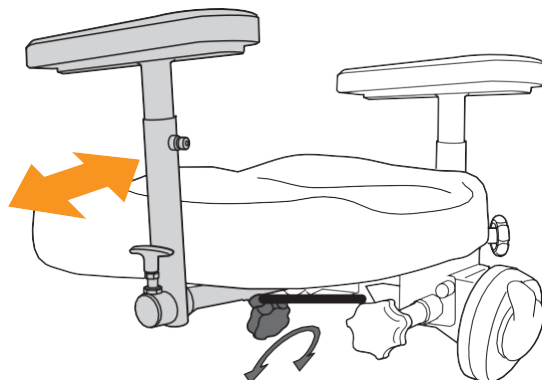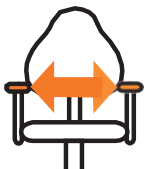ARM REST - height and width

To adjust the width between the arm supports, loosen the wheel nut (é). Adjust to desired width and tighten the wheel. Repeat on the other arm support. Tighten the wheel nuts. To adjust the height of the arm support, loosen the wheel lever (1). Adjust to desired height and tighten the wheel lever.

OPTION! ARM REST - fold back

ARM REST - fold back

It is possible to fold the arm support back wards to make sideways transfer easier or to get closer to a table. Lift up the lever and fold the arm support backwards, use your hand to lift it up where its locked automatically.

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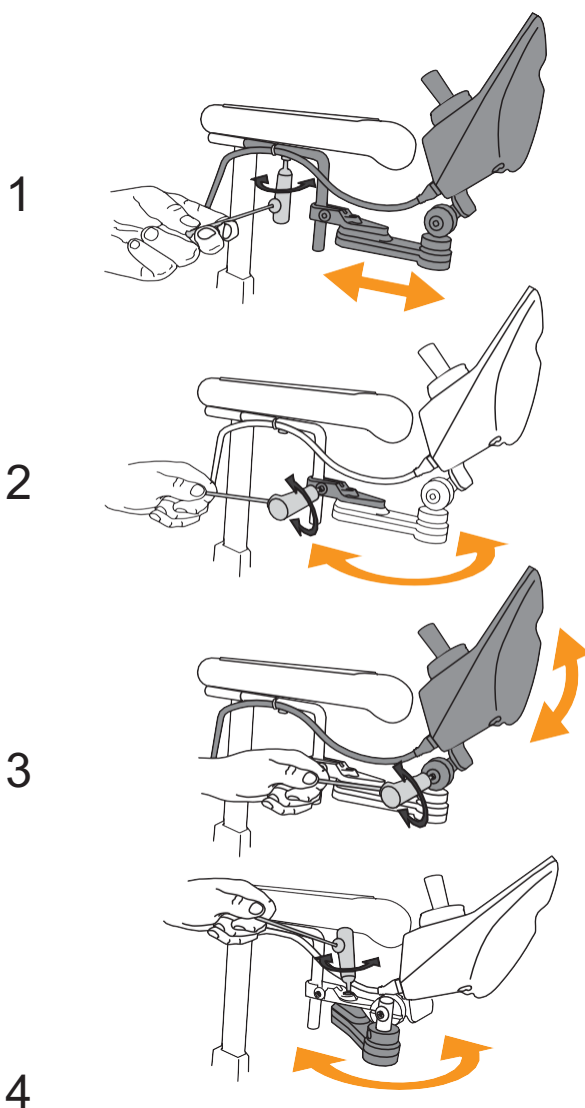
CONTROLBOX - Adjustments

The controlbox position can be adjusted by loosening the screws 1,2 and 3 with allen key nr 5. Loose screw 1 to adjust in depth and screw 2 and 3 to adjust height and angle. Adjust to desired position and tighten the screws. The controlbox can also be swung away when you would like to get close to a table i.e.

ATTENTION POINT DE
PINCEMENT!
CAUTION PINCH POINT!

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OPTION!

NECK REST - adjust

If the chair is equipped with a neck support it can be adjusted by the knob (1) to adjust to desired position. Move the knob back to the original position to lock.

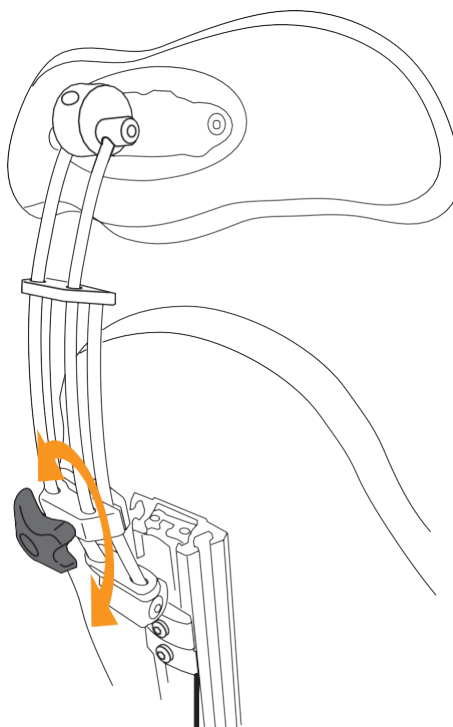
CAUTION PINCH POINT!

When the knob is opened the neck support will come loose and can rapidly fall downwards. Be careful when adjusting the neck support!

ATTENTION POINT DE PINCEMENT!
CAUTION PINCH POINT!

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English

NECK REST Comfort - adjust

The seat is equipped with a headrest will adjust it by loosening the screws (1, 2, 3) to adjust the height, depth and angle of the headrest. Lock neck support by tightening the screws.

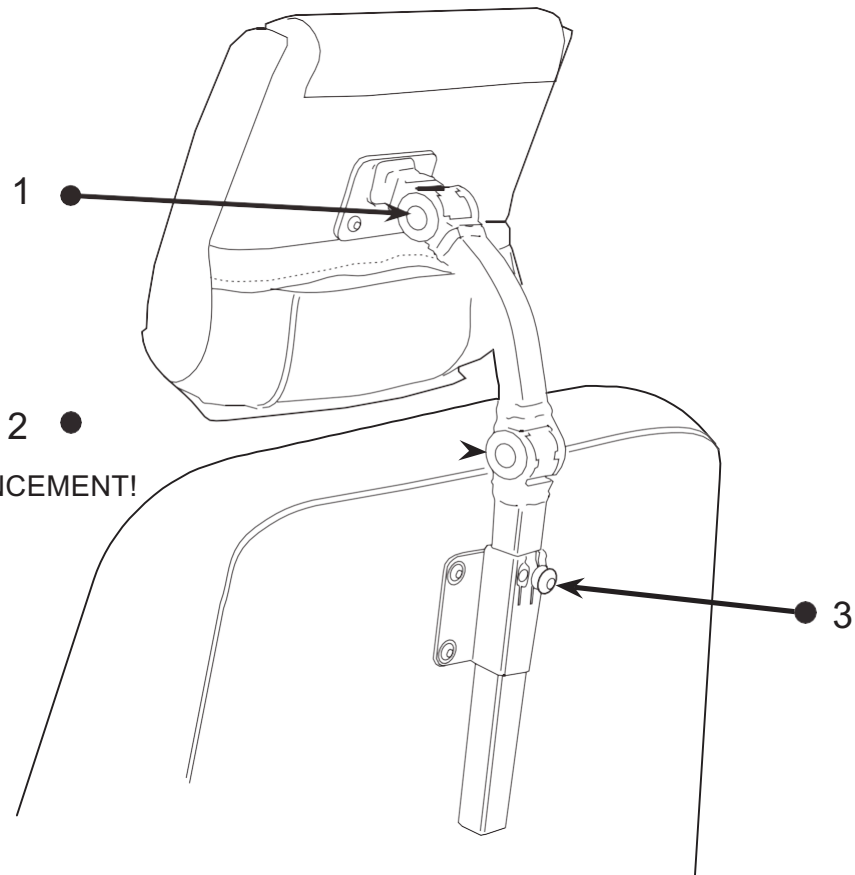
RISK OF FALLING!

NOTE! When the screws are loosen and the neck support can rapidly fall forward and downward. Be careful when adjusting the neck support!

ATTENTION POINT DE PINCEMENT!
CAUTION PINCH POINT!

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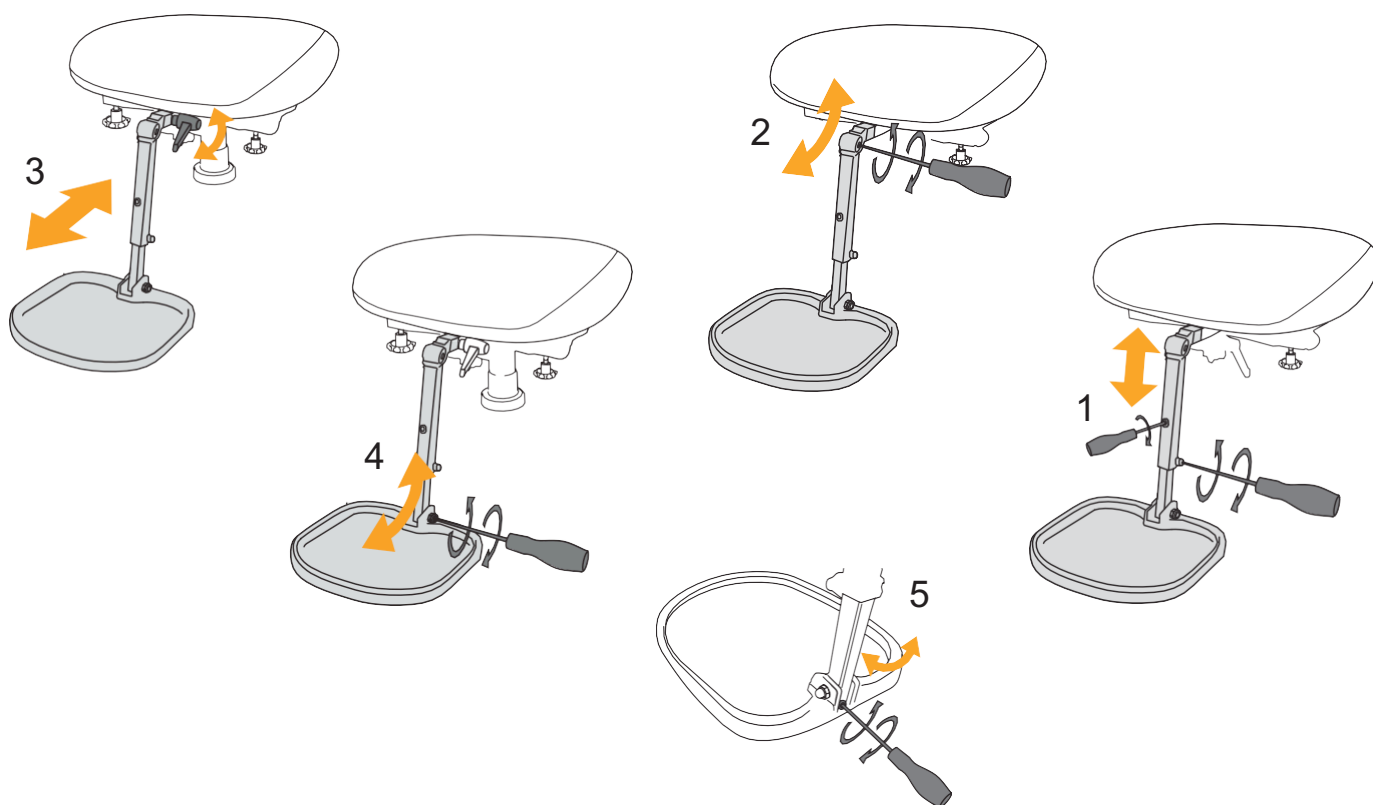
English

LEG REST - single footplate

If the chair is equipped with a single footplate you can adjust the height and lower leg length by loosening screw (1 and 2) with allen key 5 mm. Adjust to desired position and tighten the screws. To change the ankle angle, adjust the allen screw behind the footplate. (5). To adjust depth and the angle of the footplate loosen screws (3 and 4). Adjust to desired position and tighten the screws. The footplate is possible to fold up to make it easier to get in and out of the chair.

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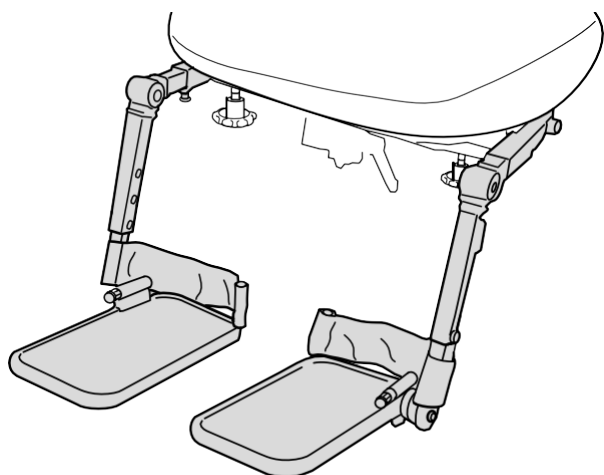
english

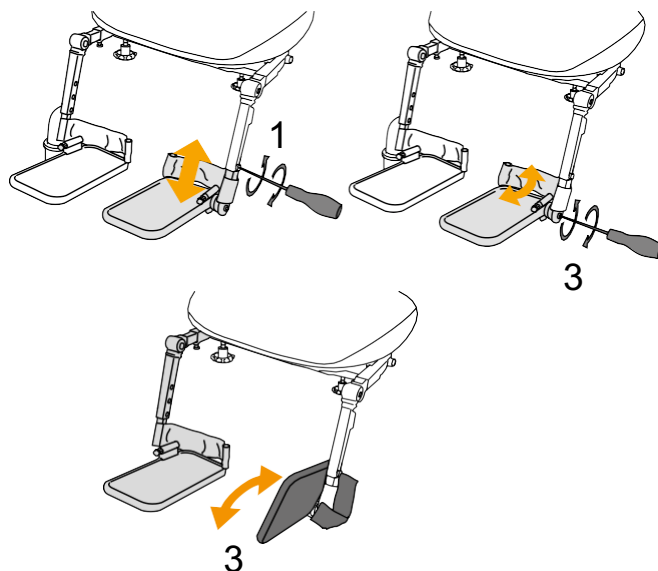
LEG REST - divided footplates leg rest - divided footplate

If the chair is equipped with divided footplates is's adjusted in height and lower leg

angle by loosening screws (1). Adjust to desired position and tighten the screws. To adjust depth and angle of the footplate loosen

screw (2). Adjust to desired position and tighten the screws. To adjust the legrest position

sideways loosen the knob. Adjust to desired position and tighten the knob (3). The footplate is possible fold up to make it easier

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LEG REST Minicross - angle adjusting
Dynamic DX, press the red button to get to the seat functions. Choose leg by moving the joystick to the right or left, until the symbol for the leg is on. Adjust the angle of the leg support forward, by moving the joystick lever forward and backward by moving the joystick lever back.

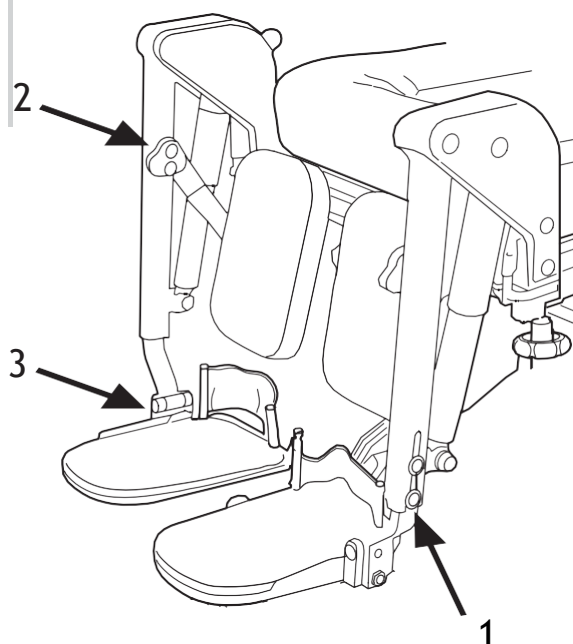
PG R-net: press the MODE button to access the seat functions. Choose leg by moving the joystick to the right or left, until the symbol for the leg is on. Adjust the angle forward, by moving the joystick lever forward and backward by moving the joystick lever back.

LEG - length adjustment

To adjust the length of the leg rest to unscrew the screws indicated (1) a turn to adjust benstödslängden. Tighten the screwjoint at the right length is set.

LEG - the foot plate angle adjustment
Loosen the bolts on the arrow (3) to adjust the angle of the footplate.

CALF - height and depth adjustment
Loosen the bolts at (2) to adjust the height and depth of the calf support.

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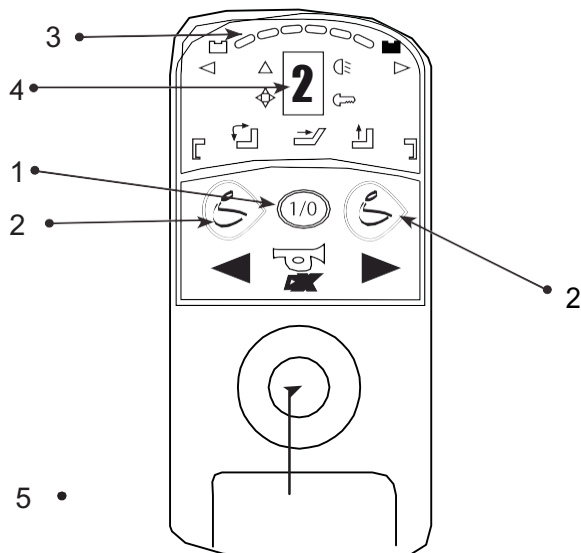
G90A

DRIVING

Push the main switch on and place yourself as comfortable as possible in the chair and let the arm you should use to control the wheelchair rest on the armrest so that your hand can grip the joystick (5) comfortable.

Start the electronics by pushing button (1), wait a couple of seconds until the battery indicator (3) has stopped twinkle.

G90A/PG R-net

Choose desired drive program (1, 2 or 3) with the buttons (2), where number 3 is the fastest. The more LEDS or higher number on the display (4) the faster the wheelchair goes.

Now lean the joystick at the direction you wish to go and the wheelchair goes there. Release the joystick and the wheelchair will stop immediately.


While driving:

- Do not push power on/off button.
- Do not start suddenly drive to opposite direction

**RISK OF INJURE!**

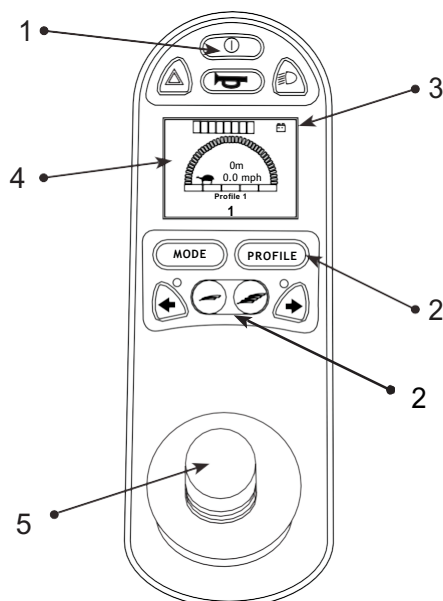NOTE! Make sure that you have a lot of free space around the wheelchair when driving the first time. Practice to drive backwards, turning and forward at different speed to get familiar how the wheelchair behave at different situations. Be careful when passing obstacles like thresholds or similar.

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DRIVE TRAINING

It is important to learn how the wheelchair works in various driving situations. Training is the key to develop your driving skills. But do not train alone.

- **Practice starts.** In the beginning it can be difficult to drive smooth without jolt. Take time to learn to drive smoothly. It is more comfortable and the risk to drive on different things decrease. Try to be soft in the hand / wrist and support your entire forearm against the armrest.

- **Practice braking** so that they become soft. To make sudden braking is not difficult, just release the joystick. Learn how long braking distance your electric wheelchair needs for stopping smoothly.

- **Practice to drive over thresholds.** Low thresholds (5-15mm). Make sure the wheels are pointed straight at the obstacle and pass it with caution. High thresholds (15-30mm), you may have to pass parallel to the obstacle.

- **Practice turns on right and left.** Notice how large floor space is needed to swing the chair a full turn around.

- **Practice to back.** Drive smoothly and slowly. Note that the wheelchair reacts in a completely different way when turning. (Release the joystick immediately), if you lose control and start over again. Lean slightly forward when reversing over a threshold.

- **Practice driving in narrow passages.** Train to drive through the doorway from the side and from the front. Train also backing into tight spaces.

- **Brake.** Move the joystick in the middle position.

- **Emergency stop.** Release the joystick. Stop distance is approx. 1m at 5km/h.

Raised seat lift and seat tilt oblique or back angle

changes the center of gravity and increases the risk of tipping! Use seat functions only on flat surfaces and always drive with great caution and at low speed!

Keep in mind that a surface that slopes sideways may affect steering of the wheelchair.

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English

DRIVING INSTRUCTION

Drive instructions for sloping and bumpy underground.

Make sure the seat lift is in its lowest position!

Do not turn or brake the wheelchair suddenly the sloping ground.

The maximum height for a threshold when driving forward 40mm / backward 30mm.


This wheelchair is designed to be used only indoor and to climb obstacles of up to 4 cm. These obstacles should be driven over the front wheels diagonally or straight to the obstacle.

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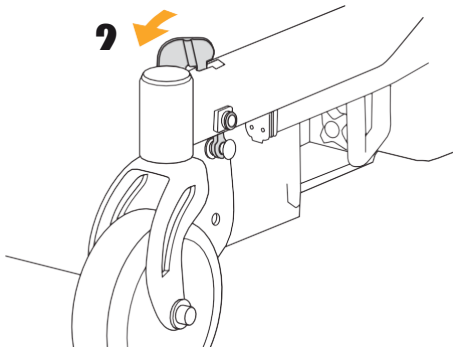
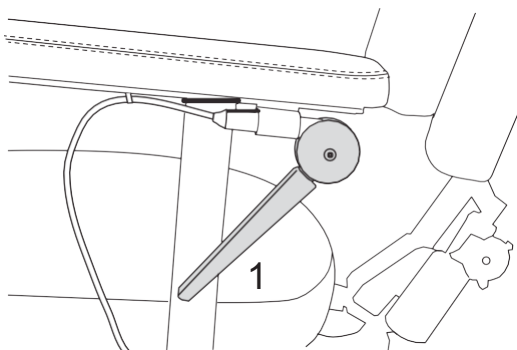
**DISENGAGEMENT OF BRAKES**


To be able to transport the wheelchair without using the electronics you need to disengage the brakes/motors. This is done by turning of the electronics and pushing the red lever placed on the chassis leg by the rear castor "forward until it's locked in forward position" (1). Now you can transport the wheelchair without motor brakes. To get back into driving you "push the red lever out of the slot" and starts the electronics. If you start the electronics when the brakes are disengaged it will cause an error code on the control box. Turn off the electronics engage the brakes and restart the electronics, the error code will disappear.

**RISK for injuries!**

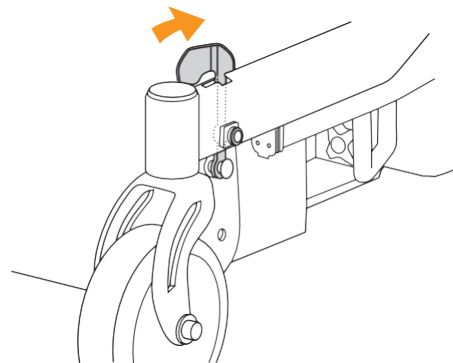NOTE! Make sure that the wheelchair is on flat ground when disengaging the brakes. Otherwise the wheelchair can start rolling away without control.

**OPTION not standard**

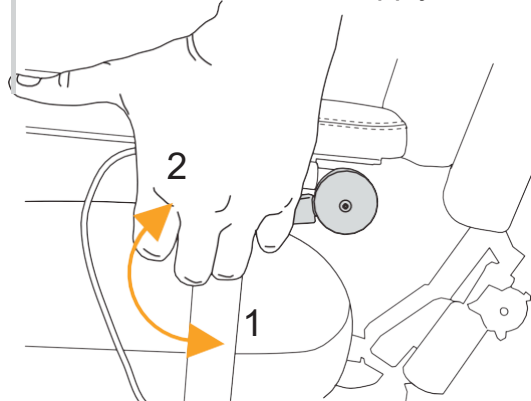1. Lift the lever to release the brake.
2. Release the lever to apply the brake.

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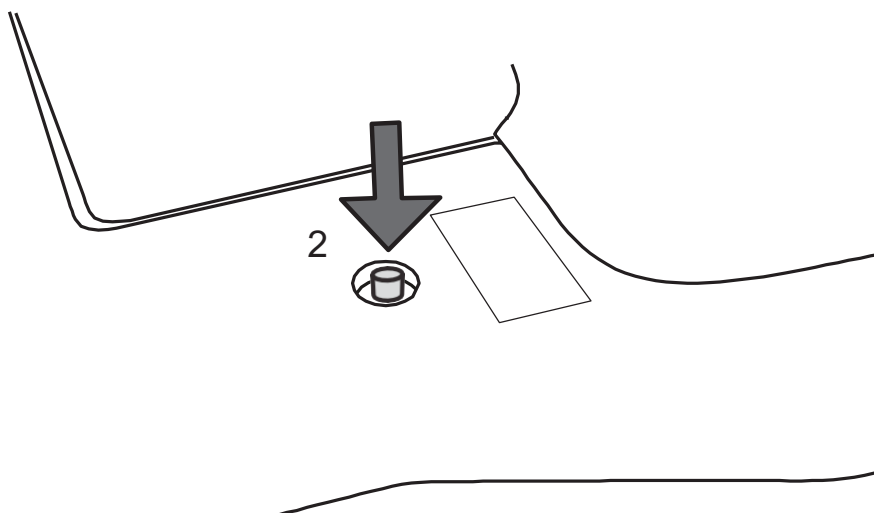
AUTOMATIC FUSE

The automatic fuse is placed on the rear cover on Flexmobil (1). The fuse will disengage if a severe electrical failure occurs on the chair. The button pops out approx. 8 mm and a white line becomes visible. The Automatic fuse can be used as a main-power switch. When the wheelchair is stored for a longer period the fuse should be disengaged.

NOTE!

When the batteries is charged the fuse must be engaged. For transport in a aeroplane the batteries usually must be disconnected and for this you can use the automatic fuse that should be disengaged.

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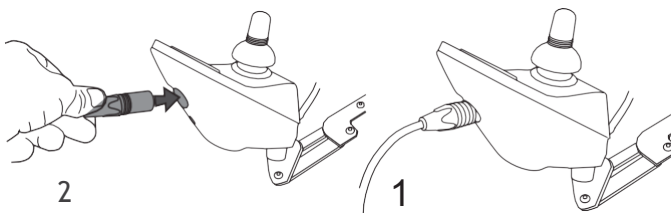
CHARGING OF BATTERIES

To keep the full power of the batteries as long as possible it is important to charge them regularly. Charge daily, if used daily. Never stop the charging until its completed. Do not charge in confined

unventilated areas due risk of gas formation. When stored the batteries should be charged once a month to prevent them to go under the critical level when the charger cannot start. Do not leave the recharging on for long continuous periods as this is not good for the batteries.

Battery indicator:

The last yellow diode goes down = max 12min remaining driving time(approx. 700m) One blinking red diode = No driving, batteries must be charged immediately.

NOTE!

If the batteries are damaged or leak, all contact with these should be avoided to prevent corrosive injuries. Contact an authorized workshop as soon as possible!

Used batteries should be handed in for recycling. Use only original batteries from Eurovema Mobility AB (Yuasa NPC24-12V 24Ah).

1. Connect the charger plug to the control box on the chair. Located in the front of the control box. See picture.
2. Connect the charger to the wall socket.
3. Check on the charger's indicator lamps that recharging has started. Information on the charger is supplied with the charger.
4. When recharging has been completed, switch off the charger or remove the plug from the wall socket and then the charger plug from the control box.

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WARRANTY

We give a 2 year warranty on our products for fabrication fault. Exceptions are fabric, wheels and batteries where we give 1 year warranty. A normal wear and tear is not accepted as a warranty. We urge our customers to use the products according to the manual

SERVICE

If the wheel chair is used frequently, i.e. every day, it ought to have service once a year. This to control that the function and security of the wheel chair is kept through all its lifetime. If service is needed contact primarily your local dealer.

CE - label

This product has the CE – mark according to the directive 93/42/EEC. The CE mark is on the label. Tested and approved 1. April 2016 according to standard EN12184:2014. Fire tested according to EN 1021 1,2:2014 (Uppholstery: Cigarette and match flame).

Manufacturing label is located on the back of lifting columns.

1. Manufacture
2. Article + Seating system (SitRite)
3. Article Number
4. Serial Number and date of manufacture
5. Class II Equipment
6. Duty Cycle
7. IP Code
8. Input Power Charger
9. Read Instructions
10. Max user weight
11. Recycle
12. CE Marking
13. Bar code

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MAINTENANCE

You get more joy out of your power wheelchair if it is served correctly. The batteries must be charged, the seat must be washed and wiped, the tires must be checked regularly and periodically the wheelchair may need have a drop of oil in their joints to not seize up. Keep Service Journal!

CLEANING:

Upholstery and Cloth: For normal cleaning, wash the upholstery with hand warm water and a mild nonabrasive soap. Use a soft cloth or brush. Before the surface dries, wipe off any water/soap residues with a clean, dry cloth. This procedure may be repeated to remove stubborn dirt or stains.

If necessary the upholders can be machine washed in 60 degrees Celsius.

Metal and plastic surfaces: For normal cleaning it is best to use a soft cloth or sponge, hand warm water and mild detergent. Wipe off metal and plastic surfaces with a soft cloth, mild detergent and hand warm water. Dry with a soft cloth. If necessary, wipe off with pH neutral disinfectant. Do not use solvents or

abrasive kitchen cleaners or aggressive chemical cleaner or cleaning fluids. These will damage the surface and structure of the material.

Never use the high-pressure washer or steam cleaner. This will damage the surface and structure of the material and could cause electronic failure.

FUNCTION:

If loose screws or loose parts is discovered somewhere on the wheelchair or a change in drive characteristic, immediate repair should take place when this could affect safety. Contact your local agent or the manufacturer for help and questions.

Be sure to tighten all screws and knobs after seat adjustments.

STORAGE:

If the wheelchair should be unused for a longer period (a month or more) it should be stored in a clean, damp free room tempered space. The batteries should be charged once a month to prevent them to go under the critical level when the charger cannot start.

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Action!

The wheelchair doesn't start?

The automatic fuse is released?
 The batteries is discharged?
 Loose cables?

Check the fuse - see the manual.
 Charge the batteries!
 Contact service!

The batteris can not be charged?

The automatic fuse is released?
 The batteries is totally discharged?
 The charger is brooken?

Check the fuse - see the manual
 Contact service!
 Contact service!

Battery indicator drops quickly despite charged batteries?

The batterier starts getting bad?
 Error in the electronics?

Contact service!
 Contact service!

The status LED is blinking on the controlbox?

Electronics has been started when brakes are disengaged?
 An electric fault has occured on the wheelchair?

Turn off and engage the brakes- see the manual!
 Contact service!

The wheelchair is running slow?

Is the right drivprogram selcted?

Select a quicker driveprogram- see the manual!

The wheelchair don't manage to climb obsticles?

A too slow driveprogram is selected?

Select a quicker driveprogram- see the manual!

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TRANSPORT

When transporting Miniflex or Flexmobil in a motor vehicle it is important that the brakes are engaged. See section "Disengagement of brakes". The chair should be secured with tie down straps.

Special brackets to be mounted on the chair are available as accessories.

The wheelchair is not designed for transport with the user seated in the chair.

You can reduce the transport size by removing the armrest, backrest and the legrest.

The batteries in the wheelchair are maintenance free and of the type GEL/AMG and approved for transport in aircrafts.


Smallest transport size:

	Flexmobil i6 Standard	Flexmobil i6 Junior
Length	80 cm	80 cm
Width	58 cm	58 cm
Height	52 cm	44 cm
Weight	97 kg	97 kg

Back-rest	5 kg
Arm-rest	2 kg
Seat	9 kg
Foot-rest	3 kg

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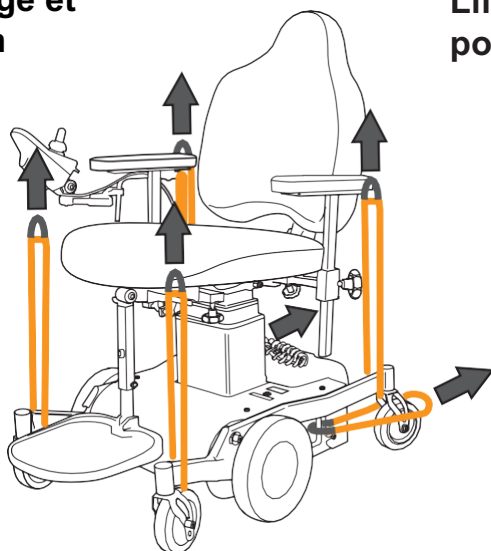Instructions for transfer of the the wheelchair user

- Prepare where the user will be transferring to.
- Ask if the user can help and tell him or her what you will do even during the transfer.
- When transferring to a wheelchair, be sure to have armrest removed or flipped backwards to avoid contact or injury.
- Remove the footrest, or swing away to prevent feet from becoming tangled when performing the wheelchair transfer.
- Position the wheelchair either at the same height or slightly lower to assist in a safe and proper wheelchair transfer.
- Depending on transfer method, having the wheelchair parallel to transfer point may also allow for a more safe and easy wheelchair transfer.
- Ensure the brakes are activated and the wheelchair is stable.


Perform the one person wheelchair transfer

- Ensure the user is utilizing a gait or transfer belt
- Stand as close to the user as possible. Be sure footing is as stable as possible.
- Proper shoes is also recommended as sandals or flats do not provide support for a safe wheelchair transfer.
- Assist the user in nearing the edge of the wheelchair.
- Be sure the users feet are under his or her body.

Picture 1. Lift on the belt while the user is leaning slightly forward and rises her/him up.

Picture 2. Keep the users weaker knee in between your own to assist and to slowly sit onto wheelchair.

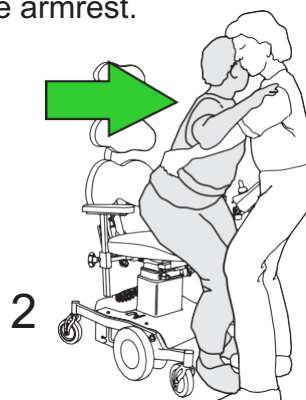- Securely position the patient on the chair and replace the armrest.

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Refurbishment and re-use of the wheelchair

This wheelchair is suitable for refurbishment and re-use. This means that, if the wheelchair is not used anymore by the first user, it can be refurbished to fit another user. Should you not be able to use the wheelchair any longer, we strongly recommend you to contact your local authorized supplier to have it picked up refurbishment and re-use.

The refurbishment of the wheelchair is only allowed to be done by an authorized supplier.

NOTE! The wheelchair must be be refurbishment according to a refurbishment guidelines of local authorized supplier. This includes the replacement of all upholstery parts, a total disinfection of the product and a complete technical check of the wheelchair and its accessories.

Warning: Dangerous products - The use of disinfectants is restricted to authorized personnel.

All parts of the wheelchair can be treated by scrubbing with a disinfectant.

Disposal of the wheelchair

We strongly recommend you to contact your local authorized supplier to take care of the disposal of your wheelchair.

If you wish to dispose of the wheelchair yourself, please enquire at your local disposal companies about the applicable local regulations

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- batteries

Used materials and sorting guidelines

1. Metals

- chassis
- wheel forks
- seat cross
- lifting column
- armrests frames
- joystick bracket
- leg-rests
- neck-rest
- backrest
- plate-cover
- electronic plate
- freewheel pedal
- rims
- shafts
- bearing
- brake-wire
- spacer
- washers
- bolts, screws and nuts

2. Burnable materials

- fabric covers
- polyurethane
- castors
- drive wheel
- knobs
- dampers
- battery cover
- cable lock
- straps
- seals
- armrests

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